



Complete with –

- o Air Bulb Lead



Description:

Typically this unit would be located beside the bath to enable the service user to call for assistance if required. The sealed 2.5m Air Bulb Lead will float in the bath so the service user can always reach the bulb without having to search for it under the water. A simple squeeze of the floating air bulb activates the pressure switch in the wall mounted unit, which generates a patient call.

The large orange “staff call” button generates the next level of call indicating further assistance is required by a member of staff, whilst the red “alarm” button indicates an emergency situation.

The green “reset” button not only cancels a call once dealt with, but if no call is active can be used to indicate “staff presence” in the service users room for regular checks without generating a call.

Optional extra's for this unit to trigger an assistance call could be –

- o ARM infrared call point upgrade to allow infrared keyfobs/pendants to be used
- o Other possible air pressure trigger devices e.g. Pneumatic pad

Levels of Call:

- o Patient Call
- o Staff Call
- o Emergency
- o Reset
- o Staff Presence
- o Low Battery

Technical Specification:

- o Reassurance LED and comfort tone
- o Factory set unit identity plus fully supervised monitoring
- o Conforms to BS6799 Class VI
- o Effective radiated power: 10mW integral aerial, approved to MPT1344 (licence exempt)

In pursuance of a policy of continued improvement, the company reserves the right to change the design and specification without prior notice



Alarm Radio Monitoring Ltd
Brunel Road
Enterprise Park
Leominster
HR6 0LX

Tel: 01568 610016
Fax: 01568 615511

E-mail:
sales@arm.uk.com

Website:
www.arm.uk.com